



DRG[®] Saliva Control Set for Saliva ELISAs (SLV-4576)

Revised 22 Nov. 2010 rm (Vers. 2.1)

USA: **RUO**

Please use only the valid version of the package insert provided with the kit.

INTENDED USE

This product is intended for use in the quality control of immunoassays using saliva samples.

This kit is intended for Research Use Only. Not for use in diagnostic procedures.

The Control Set can be used for:

SLV-4409	DHEA-S (Salivary) Elisa
SLV-3653	Estriol free (Salivary) ELISA
SLV-4780	Androstenedione (Salivary) ELISA

Safety PRECAUTIONS AND WARNINGS

Do not pipette by mouth.

Exercise the normal precautions required for handling laboratory reagents.

Since no method can offer complete assurances to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and should be disposed of accordingly.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated (2 to 8°C).

Product is stable until the expiration date printed on the external label.

Dispense only the required amount of product necessary to perform the work.

Do not return any unused material to the original vial.

UNOPENED: Store refrigerated (2 to 8°C).

Material is stable until the expiration date printed on individual vial.

Material Provided

Saliva control:

Low level 1 vial x 2 mL

Medium level 1 vial x 2 mL

High level 1 vial x 2 mL

Equipment REQUIRED BUT NOT PROVIDED

Volumetric pipette

VALUE ASSIGNMENT

Each batch of Immunoassay control is submitted to a number of reference laboratories and values are assigned from a consensus of results obtained by these laboratories. A mean value has been assigned to this lot of control using an ELISA method.