FT4 Ria

KIPB1363

FT4 Ria

en

For the In Vitro Determination of Free Thyroxine in Human Serum and Plasma.

KIPB1363

IN VITRO DIAGNOSTIC

DIAsource ImmunoAssays SA - Rue de l'Industrie 8, B-1400 Nivelles, Belgium - Tel: +32 67 88 99 99 - Fax : +32 67 88 99 96

PRINCIPLE OF THE ASSAY

The radioimmunoassay of free thyroxine (T₄) is a competition assay based on the principle of labeled antibody. Samples and calibrators are incubated with ¹²⁵I-labeled monoclonal antibody specific for T₄, as tracer, in the presence of a biotinylated analog of thyroxine (ligand) in avidin-coated tubes. There is competition between the free thyroxine of the sample and the ligand for the binding to the labeled antibody. The fraction of antibody complexed with the biotinylated ligand binds to avidin-coated tubes. After incubation, the content of tubes is aspirated and bound radioactivity is measured. A calibration curve is established and unknown values are determined by interpolation from the curve.

REAGENTS PROVIDED

All reagents of the kit are stable until the expiry date indicated on the kit label, if stored at 2-8°C. Storage conditions for reagents after reconstitution or dilution are indicated in paragraph Assay Procedure.

Kit for determination of free T₄, 100 tubes

Coated tubes for the binding of the ligand : 2 x 50 tubes (ready-to-use)

Ab	125I	¹²⁵I-labeled monoclonal antibody: one 45 mL vial (ready-to-use) The vial contains 310 kBq, at the date of manufacture, of ¹²⁵ I-labeled immunoglobulins in liquid form with bovine serum albumin, sodium azide (<0.1%; see § Precautions) and a dye.
----	------	--

CAL	N	Calibrators: five 0.5 mL vials (ready-to-use) The calibrator vials contain contain from 0 to 75 pmol/L of free T ₄ in human serum and sodium azide (<0.1%; see § Precautions). The exact concentration is indicated on each vial label. The vials must be tightly capped immediately after pipetting and stored at 2-8°C. The evaporation of the calibrator solutions from open vials may influence results of the further determination. Calibrators are verified to an internal reference calibrator.
-----	---	---

Ag	BIOT	Ligand: one 12 mL vial (ready-to-use) The vial contains a ligand solution which includes also bovine proteins and sodium azide (<0.1% see § Precautions).
----	------	---

CONTROL	Control serum: one vial (lyophilised) The vial contains T ₄ in human serum (see § Precautions). The expected values are in the concentration range indicated on the supplement. The vial must be tightly capped immediately after pipetting and stored according to § 6.1. The evaporation of the solution from open vial may influence results of the further determination
---------	--

Attention: All liquid reagents should be examined for the absence of precipitates; the antibody solution should be clear and blue-green, the calibrators may be opalescent and the ligand should be clear and colourless.

MATERIAL PROVIDED BUT NOT REQUESTED

In addition to standard laboratory equipment, the following items are required:

- Precision micropipette (25 µL).
- Semi-automatic pipets (100 µL and 400 µL).
- Vortex type mixer.
- Horizontal or orbital shaker.
- Aspiration system.
- Gamma counter set for 125 iodine.

PRECAUTIONS

- General remarks**
 - Bring all reagents to room temperature before pipeting.
 - Do not mix the reagents from kits of different lots.
 - A calibration curve must be included with each assay.
 - The correct setting of the shaker is very important for the reproducibility of the assay.
 - It is recommended to perform the assay in duplicate.
 - Each tube must be used only once
- Basic rules of radiation safety**

The purchase, possession, utilization, and transfer of radioactive material is subject to the regulations of the country of use. Adherence to the basic rules of radiation safety should provide adequate protection:

 - No eating, drinking, smoking or application of cosmetics should be carried out in the presence of radioactive materials.
 - No pipeting of radioactive solutions by mouth.
 - Avoid all contact with radioactive materials by using gloves and laboratory overalls.
 - All manipulation of radioactive substances should be done in an appropriate place, distant from corridors and other busy places.
 - Radioactive materials should be stored in the container provided in a designated area.
 - A record of receipt and storage of all radioactive products should be kept up to date.
 - Laboratory equipment and glassware which are subject to contamination should be segregated to prevent cross-contamination of different radioisotopes.
 - Each case of radioactive contamination or loss of radioactive material should be resolved according to established procedures.
 - Radioactive waste should be handled according to the rules established in the country of use.
 - This kit contains ¹²⁵I (half-life: 60 days), emitting ionizing X (28 keV) and γ (35.5 keV) radiations.
- Sodium azide**

Some reagents contain sodium azide as a preservative. Sodium azide can react with lead, copper or brass to form explosive metal azides. Dispose of the reagents by flushing with large amounts of water through the plumbing system.
- Human serum**

The materials of human origin, contained in this kit, were found negative for the presence of antibodies to HIV 1 and HIV 2, antibodies to HCV, as well as of Hepatitis B surface antigen (HBsAg). However, they should be handled as if capable of transmitting disease. No known test method can offer total assurance that no virus is present. Handle this kit with all necessary precautions.

All serum and plasma samples should be handled as if capable of transmitting hepatitis or AIDS and waste should be discarded according to the country rules.

SAMPLE COLLECTION, PROCESSING AND STORAGE

- Collect blood in dry tubes or in tubes containing heparin or EDTA, if possible after fasting.
- Separate serum or plasma from cells by centrifugation.
- Serum and plasma samples may be stored at 2-8°C, if the assay is to be performed within 48 hours. For longer storage keep frozen (<-18°C) after aliquoting so as to avoid repeated freezing and thawing.

ASSAY PROCEDURE

- Reconstitution of control serum**

The contents of the vial must be brought to room temperature before reconstitution with the volume of distilled water indicated on the vial label. Wait for 10 min following reconstitution and mix gently to avoid foaming before dispensing. Store the reconstituted solution at 2-8°C for one week or aliquoted at <-18°C until the expiry date of the kit.
- Assay procedure (see table next page)**

RESULTS

Results are obtained from the calibration curve by interpolation. The curve serves for the determination of free T4 concentrations in samples measured at the same time as the calibrators.

1. Calibration curve

The results in the package insert were calculated using a semi-logarithmic curve fit ("spline" mode) with B/T (%) or B/B₀ (%) on vertical axis and the free T4 concentration of the calibrators on the horizontal axis (pmol/L). Other data reduction methods may give slightly different results.

Total activity: 100,549 cpm				
Calibrators	free T4 (pmol/L)	cpm (n=3)	B/T (%)	B/B ₀ (%)
0	0	65,929	65.7	100
1	2.6	51,766	51.5	78.5
2	12.1	26,994	26.8	40.9
3	29.4	7,522	7.51	11.4
4	83.0	1,415	1.41	2.15

(Example of calibration curve, do not use for calculation)

2. Samples

For each sample, locate the B/T (%) or B/B₀ (%), on the vertical axis and read off the corresponding free T4 concentration in pmol/L on the horizontal axis.

To convert pmol/L to into ng/100 mL, multiply results by **0.0777**.

QUALITY CONTROL

Good laboratory practices imply that control samples be used regularly to ensure the quality of the results obtained. These samples must be processed exactly the same way as the assay samples, and it is recommended that their results be analyzed using appropriate statistical methods.

In case of packaging deterioration or if data obtained show some performance alteration, please contact your local distributor or use the following

E-mail address: tech.support@diasource.be.

EXPECTED VALUES

We recommend each laboratory to establish its own reference values. The range of values given here corresponds to clinical indications:

11.5 – 23.0 pmol/L

Remark: The following values were found on several studies on a total of 198 euthyroid patients.

PERFORMANCE CHARACTERISTICS

(for more details, see the data sheet "APPENDIX")

1. Sensitivity

1.1 Analytical sensitivity: 0.5 pmol/L

1.2 Functional sensitivity: 2.4 pmol/L.

2. Specificity

The antibody used in the immunoassay is highly specific for T4. Extremely low cross reactivities were obtained against several related molecules (D-T4, T3, T3r, etc) or therapeutic drugs that may be present in patient samples (Amiodarone etc).

3. Precision

3.1 Intra-assay

Samples were assayed in 20 times in the same series. The coefficients of variation were found below or equal to 8.3 % for serum samples.

3.2 Inter-assay

Samples were assayed in duplicate in 15 different series. Coefficients of variation were found below or equal to 7.5 % for serum samples.

4. Measurement range (from analytical sensitivity to highest calibrator):

0.5 – 75 pmol/L.

LIMITATIONS OF THE PROCEDURE

The non-respect of the instructions in this package insert may affect results significantly. Results should be interpreted in the light of the total clinical presentation of the patient, including clinical history, data from additional tests and other appropriate information.

Plasma biotin concentrations of below 40 ng/mL do not interfere with the assay. In the case of patients treated with high concentrations of biotin (>5 mg/day), blood samples must be taken at least 3 hours after the last administration of biotin.

The kit has not been validated on neonatal specimens.

Care should be taken when interpreting the results from hemolysed or icteric samples as the method has not been tested with such samples.

For assays employing antibodies, the possibility exists for interference by heterophile antibodies in the patient sample. Patients who have been regularly exposed to animals or have received immunotherapy or diagnostic procedures utilizing immunoglobulins or immunoglobulin fragments may produce antibodies, e.g. HAMA, that interfere with immunoassays.

Such interfering antibodies may cause erroneous results. Carefully evaluate the results of patients suspected of having these antibodies.

ASSAY PROCEDURE

Step 1 Additions *	Step 2 Incubation	Step 3 Counting
To coated tubes, add successively : - 25 µL of calibrators or samples and - 400 µL of tracer - 100 µL of ligand. Mix.	Incubate 60 min. at 18-25°C with shaking (>350 rpm).	Aspirate carefully the content of tubes (except the 2 tubes «total cpm»). Count bound cpm (B) and total cpm (T) for 1 min.

* Add 400 µL of tracer to 2 additional tubes to obtain total cpm.

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KIPB1363

IN VITRO DIAGNOSTIC

APPENDIX PERFORMANCE CHARACTERISTICS

Specificity

Data on cross-reactivity with several related molecules for T4 were tested by RIA using 125I-labeled T4 and separating free from bound hormone by polyethylene glycol precipitation. The results are presented in the following table :

Analog	Cross-reactivity (%)
L-thyroxine	100
D-thyroxine	33
L-3, 3', 5-triiodothyronine (T3)	0.8
L-3, 3', 5'-triiodothyronine (T3r)	10.2

Precision

Intra-assay (n=25)

Samples	A	B	C	D
Mean value (pmol/L)	9.65	13.0	25.5	27.7
CV (%)	5.92	2.48	5.24	8.23

Inter-assay (n=10)

Samples	1	2	3	4	5
Mean value (pmol/L)	7.55	15.6	21.8	27.1	81.8
CV (%)	6.24	3.41	3.56	3.01	7.48

	<u>Used symbols</u>
	Consult instructions for use
	Storage temperature
	Use by
LOT	Batch code
REF	Catalogue number
CONTROL	Control
IVD	In vitro diagnostic medical device
	Manufacturer
	Contains sufficient for <n> tests
WASH SOLN CONC	Wash solution concentrated
CAL 0	Zero calibrator
CAL N	Calibrator #
CONTROL N	Control #
Ag 1251	Tracer
Ab 1251	Tracer
Ag 1251 CONC	Tracer concentrated
Ab 1251 CONC	Tracer concentrated
	Tubes
INC BUF	Incubation buffer
ACETONITRILE	Acetonitrile
SERUM	Serum
DIL SPE	Specimen diluent
DIL BUF	Dilution buffer
ANTISERUM	Antiserum
IMMUNOADSORBENT	Immunoabsorbent
DIL CAL	Calibrator diluent
REC SOLN	Reconstitution solution
PEG	Polyethylene glycol
EXTR SOLN	Extraction solution
ELU SOLN	Elution solution
GEL	Bond Elut Silica cartridges
PRE SOLN	Pre-treatment solution
NEUTR SOLN	Neutralization solution
TRACEUR BUF	Tracer buffer
	Microtiterplate
Ab HRP	HRP Conjugate
Ag HRP	HRP Conjugate
Ab HRP CONC	HRP Conjugate concentrate
Ag HRP CONC	HRP Conjugate concentrate
CONJ BUF	Conjugate buffer
CHROM TMB CONC	Chromogenic TMB concentrate
CHROM TMB	Chromogenic TMB solution
SUB BUF	Substrate buffer
STOP SOLN	Stop solution
INC SER	Incubation serum
BUF	Buffer
Ab AP	AP Conjugate
SUB PNPP	Substrate PNPP
BIOT CONJ CONC	Biotin conjugate concentrate
AVID HRP CONC	Avidine HRP concentrate
ASS BUF	Assay buffer
Ab BIOT	Biotin conjugate
Ab	Specific Antibody
SAV HRP CONC	Streptavidin HRP concentrate
NSB	Non-specific binding
2nd Ab	2nd Antibody
ACID BUF	Acidification Buffer
DIST	Distributor

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