



## **DRG<sup>®</sup> Milk Stabilization Kit (EIA-3162)**

**Revised 7 May 2007**

**For Veterinary Use Only**

### **Name and Intended Use**

The milk stabilization kit was developed for colostrum stabilization, for use in the Duphar Aujeszky test.

With this kit we are able to stabilize colostrum for more than six months without any influence on the final test results.

### **Contents**

This kit contains two bottles (A and B) and 50 microtubes for colostrum.

### **Preparation of Samples**

The microtubes are filled with colostrum (approx. 1 ml).

5 drops of A and 5 drops of B are added to these vials of colostrum and thoroughly mixed.

After this procedure, the treated samples can be stored at 4°C.