Mini Parasep® SF Faecal Parasite Concentrator

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

Cat# 148800 Mini Parasep SF 3.3 mL w/o Fixative

4 Examination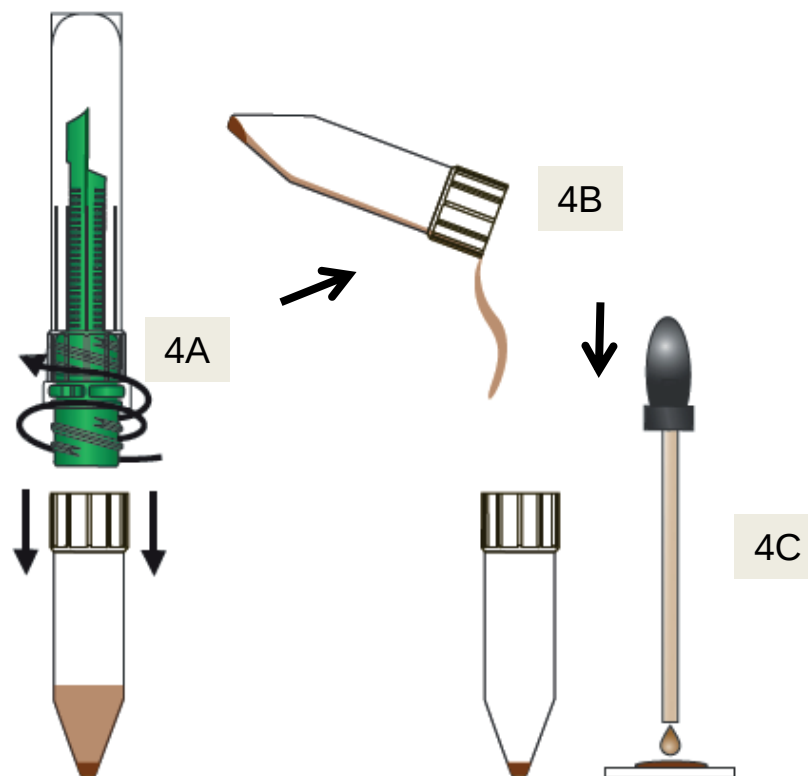

FOR FAECAL CONCENTRATION OF HELMINTH OVA AND LARVAE / PROTOZOA CYSTS AND OOCYSTS

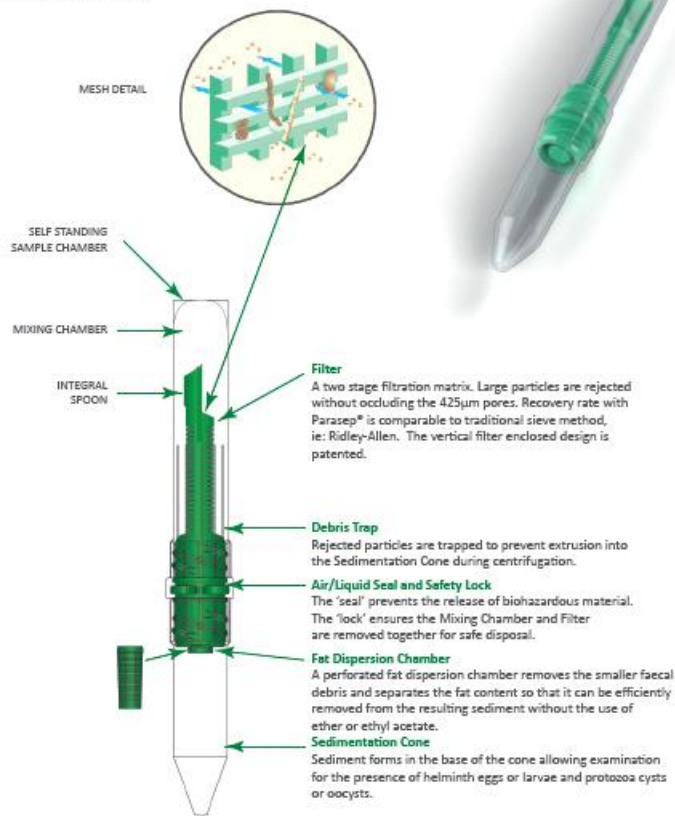Health and Safety Benefits

- Totally enclosed/sealed process
- Reduced reagent volumes
- No cleaning required
- Single use, no sample contamination
- Ready to use systems available

Performance Benefits

- Optimum sample recovery
- Enhanced sample clarity
- Rapid four step process
- Human resources optimised
- Easy patient identification
- Fits all 15ml centrifuge buckets

PARASITOLOGY
SINGLE USE IN VITRO DIAGNOSTIC DEVICE

20170920

APACOR

Mini Parasep® SF
FAECAL PARASITE CONCENTRATOR

Mini Parasep® SF
FAECAL PARASITE CONCENTRATOR

Procedure

STEP 1 - SAMPLE PREPARATION

Fresh Samples

Add a small pea sized sample (0.75g) to the reagent ready device and enclose.

STEP 2 - EMULSIFICATION

Seal the Mini Parasep® SF by screwing in the filter/sedimentation cone unit.


STEP 3 - CENTRIFUGATION

Invert Mini Parasep® SF and perform centrifugation at:

500g for ten minutes (US Gold Standard);

400g for two minutes (ARUP JCM).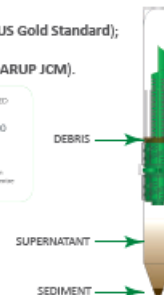

STEP 4 - EXAMINATION

Unscrew and discard the filter and mixing tube.

Decant the supernatant.

Transfer sediment to slide to perform examination where all temporary (Lugol's Iodine) and permanent staining techniques (Trichrome) can be conducted providing a universal fixative is used.

If a universal fixative is used in conjunction with this device, then molecular and EIA tests can be conducted from the sediment.

See label for storage conditions and expiry date. Please adhere to the following guidelines when handling Mini Parasep® SF. To avoid cross contamination the Mini Parasep® SF device should remain closed at all times except when introducing the sample or when retrieving the final concentrated sample for examination.

Mini Parasep® SF is available empty or reagent ready
Please ask for details

Products can be ordered direct from Apacor or from an appointed distributor
Visit our website for all the latest information www.apacor.com or email on: sales@apacor.com

UNIT 5 SAPPHIRE CENTRE
FISHPOWDS ROAD, WORKINGHAM
BERKSHIRE, RG41 3GL, ENGLAND
TEL: +44 (0)118 979 5566
FAX: +44 (0)118 979 5186

APA170 - V1 01/2016